EN

INSTRUCTIONS FOR USE

DESCRIPTION

Electroconductive contact medium for electrodes, with lithium salts, water soluble, for the acquisition of bio-signals.

INDICATIONS FOR USE

Cogel Lithium One% (Comedical) with very high conductivity is recommended for electroencephalography (EEG), electromyography (EMG), and evoked potentials (PE) measurements.

WARNINGS

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State in which the user and/or patient is established.

PRECAUTIONS

- For external use only on intact, unbroken skin;
- discard product if package is damaged or evidence of contamination is present;
- to avoid contamination of product, the tube container tips should not come into contact with patient, staff or instruments.

DIRECTIONS FOR USE

Refer to the instructions for use provided by the manufacturer of the electromedical device being used for information on how Cogel Lithium One% should be used with their product. Cogel Lithium One% is water soluble, and can be removed using a moistened soft cloth. Clean the sensing electrodes with warm water after use.

HANDLING

In case of contact with eyes, rinse immediately with plenty of water. If necessary, consult a doctor. Remove all product residue carefully from the skin. For more information consult the material Safety Data Sheet. Cogel Lithium One% is non-flammable and non-corrosive.

STORAGE

Protect product from direct sunlight. Store within the indicated temperature range.

DISPOSAL

Follow all federal, state, and local regulations for non-hazardous waste disposal.

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